

TECHNICAL DATA SHEET



UNDERWEAR

SENI ACTIVE SUPER



I. PRODUCT DESCRIPTION

Seni Active Super disposable underwear is designed for moderate and heavy incontinence needs. Top and bottom layers are made of soft non-woven. Absorbent core is made of cellulose pulp combined with superabsorbent beads. Contains distribution layer (EDS). Anti-leakage protection is provided by side standing gathers along absorbent core and leg cuffs with elastic yarn in groin area. Wetness indicator – black, printed inscription that fades when exposed to wetness. Vapour permeable outer layer on the whole surface. Z-fold tape in back area to be wrapped around the underwear after use.

II. RAW MATERIALS

Cellulose, hydrophilic nonwoven, hydrophobic nonwoven, distribution layer (EDS), vapour permeable film, elastic yarn, hot-melt adhesives, superabsorbent, Z-fold tape.

III. DIMENSIONS*

Size	Total length [mm]	Length of absorbent core [mm]	Total width [mm]	Width of absorbent core [mm] (front – crotch – back)	Suggested waist/hips girth [cm]
Small	770	480	530	140-90-140	55-85
Medium	810	490	660	150-90-150	80-110
Large	870	510	770	160-100-160	100-135
Extra Large	920	530	890	160-120-160	120-160

* tolerance of dimensions: +/- 10 mm.

IV. PROPERTIES

Size	Weight [g] no less than	Absorbency [g] no less than (ISO 11948-1)	Retention [g] average (NAFC)
Small	60,0	1200	510
Medium	65,3	1400	570
Large	75,5	1500	610
Extra Large	84,2	1600	670

V. BASIC LOGISTIC DATA

Size	Code	Package [pcs]	Packages per box	Boxes per pallet	Box type
Small	SE-096-SM10-AS1	10	8	24	carton
Medium	SE-096-ME10-AS1	10	8	24	carton
Large	SE-096-LA10-AS1	10	8	24	carton
Extra Large	SE-096-XL10-SU1	10	6	28	carton

The disposable underwear must be placed on wooden pallet, covered by PE sheets from the top and PE sheets or cardboard divider sheet from the bottom and wrapped together with PE stretch film.

VI. RECOMMENDED STORAGE CONDITIONS

It is recommended to store disposable underwear in a dry and well ventilated warehouse at stable temperature between 5°C and 35°C, at humidity between 25% and 75% and in their original packagings. The packagings must remain intact without openings in stretch film in order to avoid dust, odour and/or other contamination.

VII. OTHER

Medical device marked with CE, class I.

Validity: 60 months.

Issued by: Justyna Dziwulska, R&D Specialist

Approved by: Tomasz Przybylski, Deputy Director of
Production and Innovation of TZMO Group

Justyna Dziwulska

Specjalista ds. technologicznych

Date of issue: 27.04.2021